# INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

## 4-Hole Pyramid Adapters

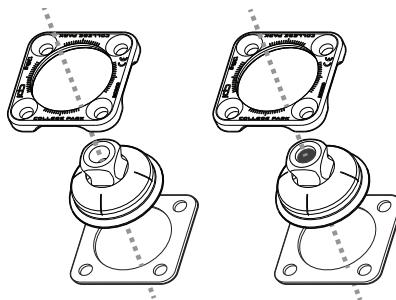
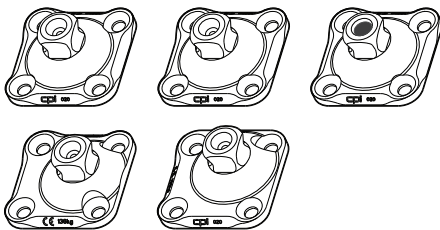
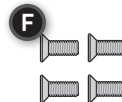
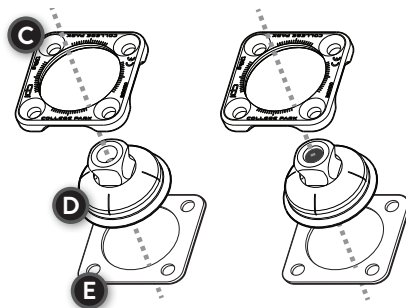
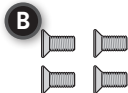
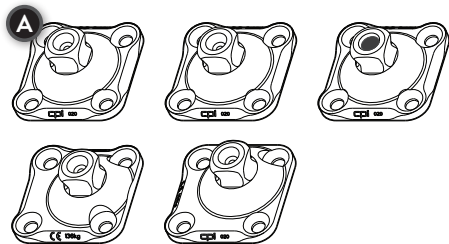
FIGURE 1

## WARRANTY INSPECTION/ MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the Warranty Inspection schedule below.

*High patient weight and/or impact level may require more frequent inspections. Prosthetic components should be inspected after the first 30 days of use. We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.*

WARRANTY INSPECTION SCHEDULE FOR ENDOSKELETAL COMPONENTS: 30 DAYS, THEN ANNUALLY.

### TECHNICAL SPECIFICATIONS

	4-Hole Pyramid Adapter, Standard, Ti	4-Hole Pyramid Adapter w/ Pin-Lock Hole, Standard, Ti	4-Hole Pyramid Adapter, Rotatable, Ti	4-Hole Pyramid Adapter w/ Pin-Lock Hole, Rotatable, Ti	4-Hole Pyramid Adapter, Standard, SS	4-Hole Pyramid Adapter, Axial Offset, Ti	4-Hole Pyramid Adapter, Diagonal Offset, Ti
<b>Part No.</b>	<b>4HPA T</b>	<b>4HPH T</b>	<b>4HPAR T</b>	<b>4HPHR T</b>	<b>4HPA S</b>	<b>4HPA AO T</b>	<b>4HPA DOT</b>
<b>Material</b>	Titanium	Titanium	Titanium	Titanium	Stainless Steel	Titanium	Titanium
<b>Build Height</b>	7 mm (0.28 in)	7 mm (0.28 in)	9.5 mm (0.37 in)	9.5 mm (0.37 in)	7 mm (0.28 in)	7 mm (0.28 in)	7 mm (0.28 in)
<b>Offset</b>	--	--	--	--	--	7 mm (0.28 in)	7 mm (0.28 in)
<b>Assembly Weight</b>	46 g	44 g	77 g	74 g	75 g	55 g	64 g
<b>Patient Weight Limit</b>	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	181 kg (400 lbs)	136 kg (300 lbs)	136 kg (300 lbs)
<b>Warranty</b>	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited
<b>Torque (Fasteners)</b>	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)

### GUIDELINES

**Non-Rotatable Adapters**

M6 x 14mm fasteners included are for use with non-rotatable adapters.  
Use of longer fasteners may be required for rotatable adapters.

**Rotatable Adapters**

M6 x 16mm fasteners are for use with College Park rotatable adapters.  
Use of shorter fasteners may be required for non-rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

## TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

**LIABILITY**

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

**CAUTION**

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

**CAUTION**

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident\* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

\*‘Serious incident’ is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following: (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient’s, user’s, or other person’s state of health, (c) a serious public health threat.

**COMPLIANCE**

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity this may correspond to 3-5 years of use.

**ISO 10328 - LABEL**

PART NUMBER	WEIGHT LIMIT (KG)	LABEL TEXT
4HPA T	136	ISO 10328-P7-136 kg
4HPAR T		
4HPH T		
4HPHR T		
4HPA DOT		
4HPA AO T	181	ISO 10328-P8-181 kg
4HPR S		

**ISO 10328 - “P” - “m”kg\*)** 

*\*) Body mass limit not to be exceeded!*

*For specific conditions and limitations of use, see intended use section of manufacturer's written instructions.*

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©2025 College Park Industries, Inc. All rights reserved.