INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

Threaded Adapters and Receivers





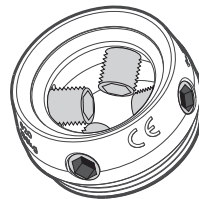
FIGURE 1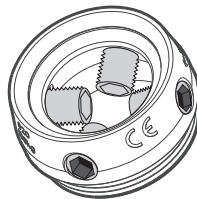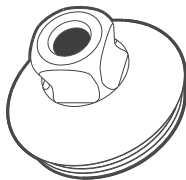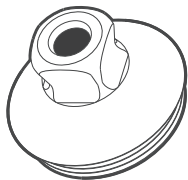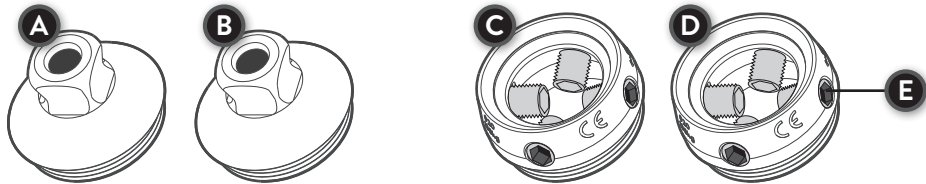

INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

Threaded Adapters and Receivers

PRODUCT DESCRIPTION

This endoskeletal component consists of a threaded pyramid adapter or receiver (titanium or stainless steel). The threaded pyramid receiver includes four set screws.

KEY COMPONENTS *(Figure 1)*

Threaded Pyramid Adapter

- A. Titanium B. Stainless Steel

Threaded Pyramid Receiver

- C. Titanium D. Stainless Steel

- E. (4) M8 x 14mm Set Screws

This diagram is to help familiarize you with the unique parts of threaded adapters and receivers, each are endoskeletal components. These parts are referenced in the instructions and used when speaking with a technical service representative.

INTENDED USE

Endoskeletal Components are intended for aligning and attaching feet, ankles, and knees on lower limb prosthesis users.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

WARNING

- Torque evenly to specified settings.
- Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.

GUIDELINES

M8 x 14mm set screws with nylon patch are included with the adapter. Loctite® is not required. M8 x 14mm set screws with nylon patch are included with the adapter. Loctite® is not required.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*'Serious incident' is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

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©2021 College Park Industries, Inc. All rights reserved.