
ICON

Icon Knee Pad Replacement Kit and Battery Replacement Instructions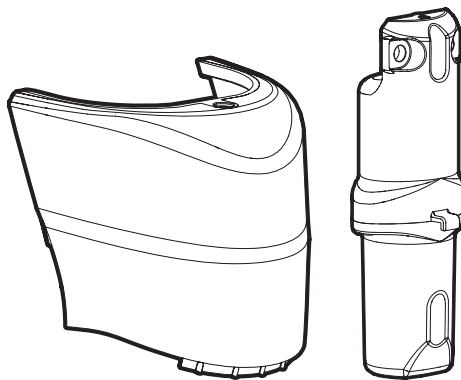

college park

TECHNOLOGY *for the* HUMAN RACE

*technical
instructions*

FIGURE 1


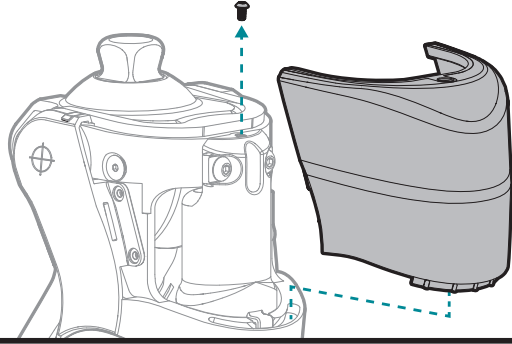
FIGURE 2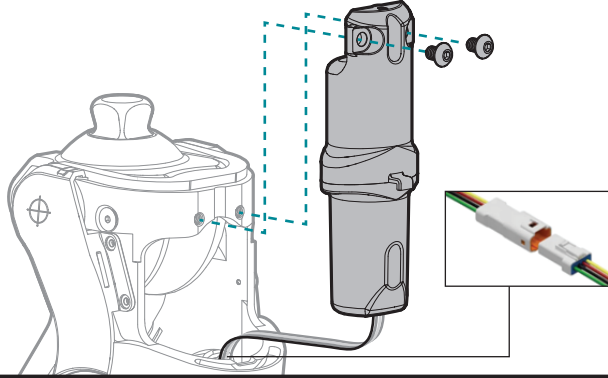

LANGUAGES

EN-English	4
AR-العربية	5
BG-Български	6
DA-Dansk	7
DE-Deutsch	8
EL-Ελληνικά	9
ES-Español	10
FI-suomi	11
FR-Français	12
HR – Hrvatski	13
HU-Magyar	14
IT-Italiano	15
JA-英語	16
KO-Kugo	17
NL-Nederlands	18
NO-Norsk	19
PL-Polski	20

PT-Português	21
PT-BR-Português do Brasil	22
RO-Română	23
RU Русский	24
SL-Slovensko	25
SV-Svenska	26
TR-Türkçe	27
UK-Українська	28
ZH-中文	29

KNEE PAD REPLACEMENT INSTRUCTIONS

PACKAGE CONTENTS

(1) Icon Knee Pad | (1) M2 x 4mm Fastener | (1) T6 Torx Driver

1. Remove M2 x 4mm fastener on the top of the knee pad and remove knee pad (*FIGURE 1*).
2. To reinstall, replace the knee pad and insert M2 x 4mm fastener (finger-tighten).
3. If replacing the battery, refer to battery replacement instructions (below).

BATTERY REPLACEMENT INSTRUCTIONS

PACKAGE CONTENTS

(1) Icon Battery Cartridge
(1) Icon Knee Pad Replacement Kit
(2) M4 x 6mm Fasteners

RECOMMENDED TOOLS

(1) 2.5 mm Hex Driver
(1) Small Flat Head Screwdriver

The battery can be replaced by the clinician if it no longer holds a charge.

1. Remove M2 x 4mm fastener on the top of the knee pad and remove knee pad (*FIGURE 1*).
2. Use a small flat head screwdriver to separate the waterproof connector.
3. Remove two M4 x 6mm fasteners holding battery in place, then remove the battery (*FIGURE 2*).
4. To reinstall, replace battery and insert M4 x 6mm fasteners (finger-tighten).
5. Reconnect the waterproof connector.
6. Replace knee pad and insert M2 x 4mm fastener (finger-tighten).

WARRANTY

The Icon Knee Pad Replacement Kit and Battery are included under the same warranty duration as the Icon Knee. Refer to College Park Limited Warranty – Microprocessor Knees for more info (available at college-park.com).

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©2024 College Park Industries, Inc. All rights reserved.