soleus®

spring into action

*amortecimento em ação • Werden Sie aktiv • salte a la acción
إمكانية القفز • ressort actionné • passe à ação
станьте активным • 马上行动*

FEATURING

*Integrated Spring
Technology®*

college park


TECHNOLOGY *for the* HUMAN RACE

technical instructions

*instruções técnicas • technische anleitung
instrucciones técnicas • التعليمات الفنية
Instructions techniques • Instruções técnicas
Технические инструкции • 技术说明书*

TABLE OF CONTENTS

EN	Technical Instructions	1 - 3
BR	Instruções Técnicas	4 - 6
DE	Technische Anleitung	7 - 9
ES	Instrucciones Técnicas	10 - 12
EU	التعليمات الفنية	13 - 15
FR	Instructions Techniques	16 - 18
PT	Instruções Técnicas	19 - 21
RU	Технические инструкции	22 - 24
ZH	技术说明书	25 - 27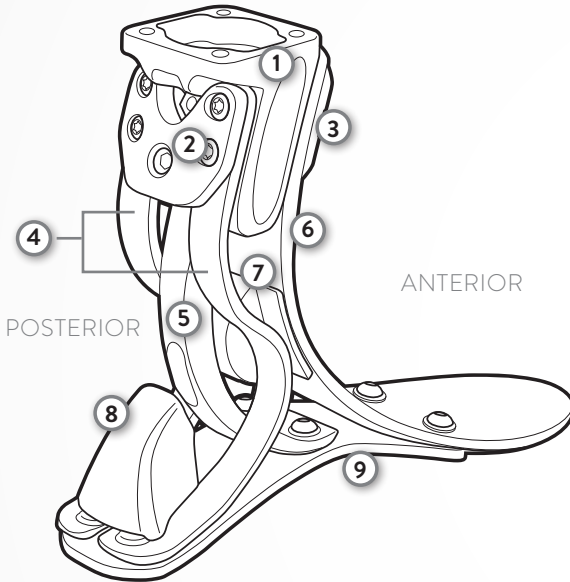

This diagram is to help familiarize you with the unique parts of the Soleus. These parts are referenced in the instructions and used when speaking with a technical service representative.

KEY COMPONENTS

1. Housing
2. Heel Spring Plate
3. Toe Spring Plate
4. Heel Springs
5. Mid Spring
6. Toe Spring
7. Mid Bumper
8. Heel Bumper
9. Belting Material

PACKAGE CONTENTS

- (1) Soleus Foot
- (1) Foot Shell
- (1) Cosmetic Attachment Plate
- (1) CPI Sock
- (1) 4-Hole Fastener Kit

TOOLS REQUIRED

- (1) 4 mm Hex Key

RECOMMENDED

- Foot Horn

MOUNTING

Use only high quality proximal endoskeletal components. Use of mounting fasteners other than those provided will void the warranty.

ASSEMBLY AND DISASSEMBLY (for sock replacement)

Use the Foot Horn to don and doff the foot shell. Remove the CPI Sock and replace as needed. Any further disassembly or modification of components will void the warranty.

STATIC ALIGNMENT

For optimal function have the patient's weight balanced between the forefoot and heel. The load line divides the foot at 1/3 heel lever.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE
Firmer Toe Response	Plantarflex the Soleus or move load line posterior
Softer Toe Response	Dorsiflex the Soleus or move load line anterior
Firmer Heel Response	Dorsiflex the Soleus or move load line anterior
Softer Heel Response	Plantarflex the Soleus or move load line posterior

MOUNTING FASTENER GUIDELINES

Fastener length will vary with adapter choice. M6 x 16mm fasteners are for use with rotatable adapters, while M6 x 14mm fasteners are for use with non-rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

CAUTION

- Apply Loctite® 242® on all 4- hole fastener threads.
- Torque evenly to 12 N·m (8.8 ft-lbs).

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure.

EXPOSURE TO MOISTURE

To maintain the lifespan of the Envirosell® and CPI sock, always dry foot thoroughly after exposure to moisture. Rinse and dry if exposed to salt water or chlorinated water.

WARNING

Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.

GAIT MATCHING® GUIDELINES

Contact College Park Technical Service if you have questions about modifying the gait match.

WARRANTY INSPECTION/ MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. We recommend you visually inspect the following applicable parts for excessive wear and fatigue at each warranty inspection.

- Composites and Adapter
- CPI Sock
- Foot Shell

WARRANTY INSPECTION SCHEDULE FOR SOLEUS: ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST).


After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

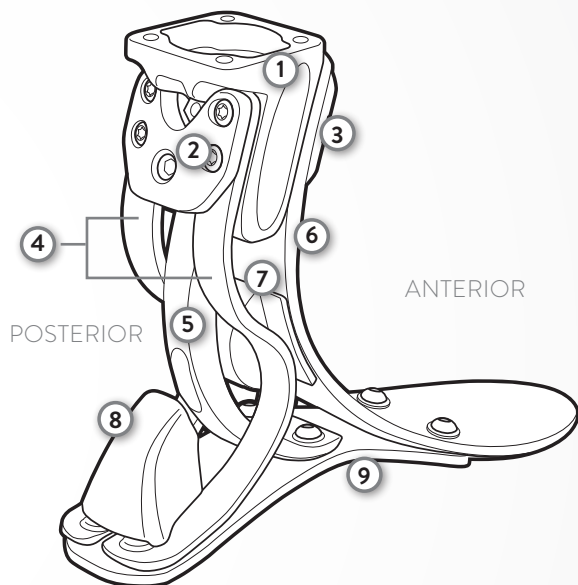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©2019 College Park Industries, Inc. All rights reserved.