technical instructions

التعليمات الفنية · Teknisk vejledning · Technische Anleitung
Τεχνικές οδηγίες · Instrucciones técnicas · Tekniset ohjeet
Instructions techniques · הוראות טכניות · Istruzioni tecniche
技術説明書 · 기술 지침서 · Technische instructies
Tekniske instruksjoner · Instrukcje techniczne · Instruções técnicas
Instruções técnicas · Технические инструкции · Technické pokyny
Tekniska anvisningar · Teknik Talimatlar · 技术说明

FIGURE 1


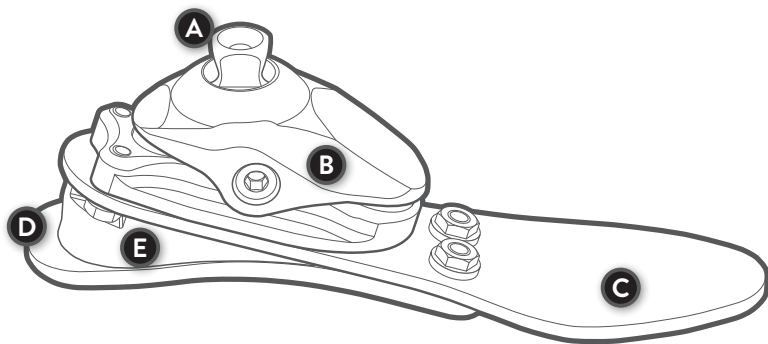
FIGURE 2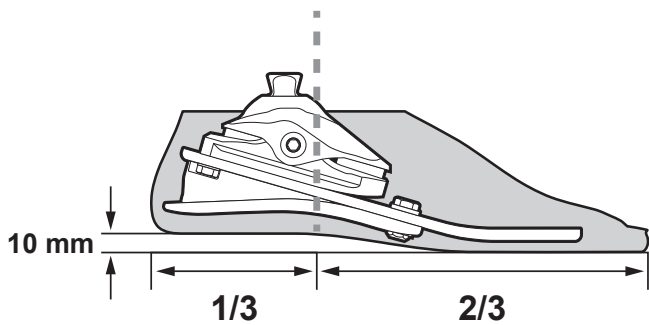

PACKAGE CONTENTS

- (1) Terrain iW foot
- (1) Foot Shell
- (1) CPI Sock

TOOLS REQUIRED

- 4mm Hex Key

TOOLS RECOMMENDED

- (1) Foot Horn

This diagram (Figure 1) is to help familiarize you with the unique parts of the Terrain iW. These parts are referenced in the instructions and used when speaking with a technical service representative.

Key Components, FOOT (Figure 1)

- | | |
|--|---|
| <ul style="list-style-type: none"> A. Integrated Pyramid C. Toe Spring E. Permanent Wedge • CPI Sock (not shown) | <ul style="list-style-type: none"> B. Ankle-Housing Assembly D. Heel Spring • Foot Shell (not shown) |
|--|---|

PRODUCT DESCRIPTION

This prosthetic foot device is constructed with an integrated pyramid, ankle-housing assembly, two composite springs, and heel wedge. The toe spring is secured to the housing and heel spring with fasteners.

INTENDED USE

The Terrain iW is a prosthetic foot designed to replace one or more functions of the biologic human foot.

⚠ INDICATIONS:

- Lower limb amputations

⚠ CONTRAINDICATIONS:

- None known

⚠ PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

TECHNICAL SPECIFICATIONS

FOOT SIZE	WEIGHT LIMIT	BUILD HEIGHT	FOOT WEIGHT*
21-24 cm	220 lbs / 100 kg	2.5 in / 6.4 cm	645 g
25-26 cm	300 lbs / 136 kg	2.7 in / 6.9 cm	
27-30 cm		2.8 in / 7.1 cm	

*26cm foot w/shell

GAIT MATCHING® GUIDELINES

The gait match determines the firmness of the foot based on the user's specifications (foot size, patient weight, and impact level).

FIRMNESS CATEGORIES

Refer to the chart below to determine the correct firmness category.

Note: Incorrect category selection may result in poor device function. Contact College Park Technical Service if you have questions about category selection.

WEIGHT LBS	0-140	141-180	181-220	221-300
WEIGHT KG	0-63	64-81	82-100	101-125
SIZE CM	21-30		26-30	
Low-Moderate Impact	1	2	3	4

ASSEMBLY AND DISASSEMBLY *(for sock replacement)*

Use the Foot Horn to don and doff the foot shell. Remove the CPI sock and replace as needed. Any further disassembly or modification of components will void the warranty.

ENDOSKELETAL MOUNTING

Use only high quality proximal endoskeletal components.

STATIC ALIGNMENT

For optimal function, balance the patient's weight evenly between the heel and toe. *(Figure 2)*

- The Terrain iW was designed with a 3/8" (10.0 mm) heel rise.
- The load line divides the foot at 1/3 heel lever and 2/3 toe lever.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE	COMPONENT CHANGE
Firmer Toe Response	Plantarflex the Terrain iW or move load line posterior	---
Softer Toe Response	Dorsiflex the Terrain iW or move load line anterior	---
Firmer Heel Response	Dorsiflex the Terrain iW or move load line anterior	Add heel wedge
Softer Heel Response	Plantarflex the Terrain iW or move load line posterior	Remove heel wedge

**To install or remove heel wedge, refer to the Horizon Wedge Kit Instructions.*

WARNINGS

WARNING

- Do not expose this product to corrosive materials, salt water or pH extremes.
- Any further disassembly or modification of components will void the warranty.

Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.

RESIDUAL RISK STATEMENT

NOTICE OF RESIDUAL RISK

During fitting process, ensure that CPI sock does not become pinched between foot and endoskeletal componentry.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. Soft component wear depends on the patient weight, impact level and environment. We recommend you inspect the following applicable parts for excessive wear and fatigue at each warranty inspection and replace as needed.

- Composites and Adapters
- Foot Shell
- CPI Sock
- Heel Wedge (if applied)

WARRANTY INSPECTION SCHEDULE FOR THE TERRAIN iW: SIX MONTHS, THEN ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*'Serious incident' is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following: (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles. Depending on patient activity this may correspond to 2-3 years of use.

ISO 10328 - LABEL

FOOT SIZE	WEIGHT (KG) LOW IMPACT	LABEL TEXT
21-24 cm	100	ISO 10328-P5-100 kg
25-30 cm	136	ISO 10328-P6-136 kg

ISO 10328 - "P" - "m"kg*)

*) *Body mass limit not to be exceeded!*
For specific conditions and limitations of use, see intended use section of manufacturer's written instructions.

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©2024 College Park Industries, Inc. All rights reserved.