AURA™

polycentric knee

college park
TECHNOLOGY *for the* HUMAN RACE

user guide

THANK YOU FOR USING THE COLLEGE PARK AURA KNEE!

Your prosthetist will instruct you on the proper use of the knee. If you do not understand the knee or have any questions about using it, please ask your prosthetist.

CARE FOR YOUR COLLEGE PARK AURA KNEE

If there is no cosmetic cover applied, you may clean the College Park Aura Knee by wiping it with a soft, lint-free cloth. Never apply harsh chemicals.

USE IN WATER

- The Aura Knee has been approved for use in water.
- The knee should be switched into lock mode when using it in or near water.
- Use caution when walking on wet surfaces.
- Always dry knee thoroughly after exposure to moisture. Rinse and dry if exposed to salt water or chlorinated water.

⚠️ WARNINGS

- The knee is not intended for kneeling or squatting.
- Avoid pinching hazards! Do not place fingers near the flexing area of the knee.
- Always report any concerns to your prosthetist immediately, including but not limited to: noise, sudden loss of function, brake release sticking, etc.
- Never attempt to adjust the College Park Aura Knee yourself. Only a trained professional should make adjustments to the College Park Aura knee.
- Do not disassemble the knee. Your prosthetist can contact College Park to arrange a repair or replacement.
- Contaminants such as dirt and the use of lubricants or powder may affect the function of the knee and lead to failure.
- Do not use compressed air to clean the knee as it can push dirt inside of the knee.

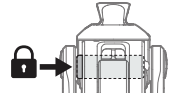
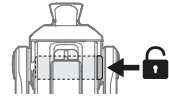
Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.

USING THE COLLEGE PARK AURA KNEE

The College Park Aura knee has two functional modes. Before you begin using the knee, your prosthetist will determine the setting that is most appropriate for you and will activate that mode.

Normal Mode	Lock is disabled; Knee utilizes polycentric geometry for stability during stance.
Locking Mode	Lock is enabled; Allows for manual locking and unlocking of knee

USING THE MANUAL LOCK FEATURE

	<p>Locking the Knee Push the lock button from the left side.</p>
	<p>Unlocking the Knee Push the lock button from the right side. The knee will now be in normal mode.</p>

USING THE EXTERNAL LOCK KIT

If the external lock kit has been installed, there will be a dual function lever attached to your prosthesis so you can control when it should be locked or unlocked.

	<p>Locked Position <i>Standing/ Walking</i> The knee will not flex.</p>
	<p>Lock Release Position Pull upward on the lever only half-way, without passing the detents. Releasing the lever will return it to the locked position.</p>
	<p>Unlocked Position <i>Deactivates the lock function temporarily</i> Pull upward on the lever all the way, passing the detents. There will be a “click” sound and feeling. Push down on the lever to return to the locked position.</p>

TO SIT DOWN

Pull upward on the dual function lever to allow the knee to flex.

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©2024 College Park Industries, Inc. All rights reserved.