
college park

STERLING

GEL LINER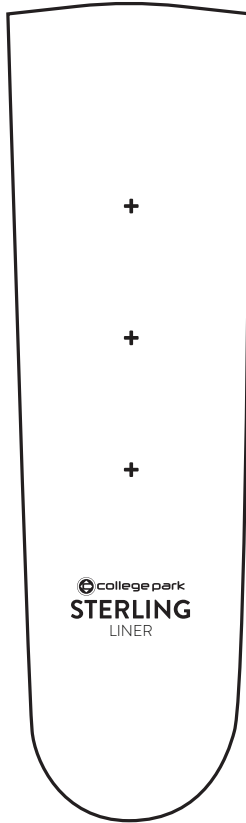

college park

TECHNOLOGY *for the* HUMAN RACE

*technical instructions,
user guide, and warranty*

FIGURE 1

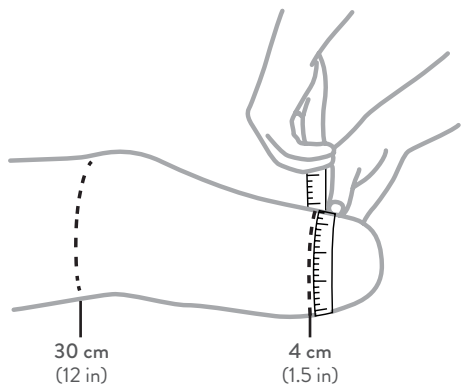
FIGURE 2

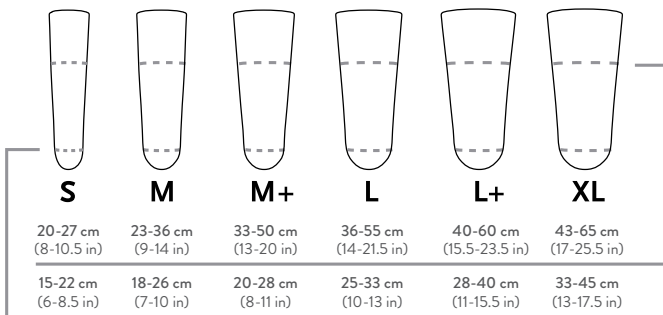
FIGURE 3

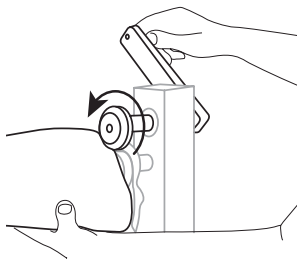
FIGURE 4

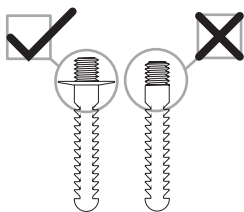
FIGURE 5

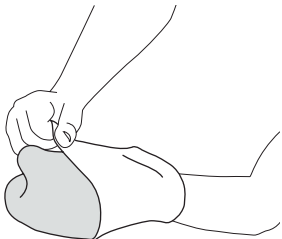
FIGURE 6

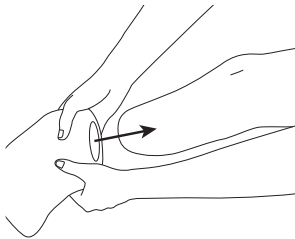
FIGURE 2

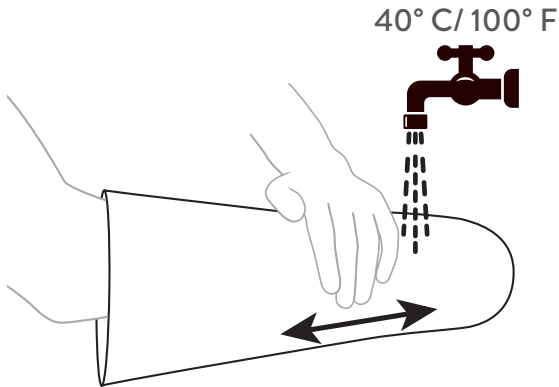
FIGURE 9

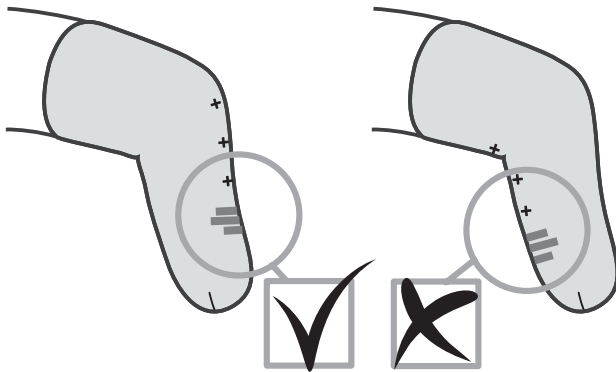
FIGURE 7


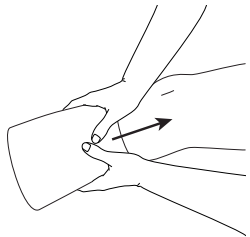
FIGURE 8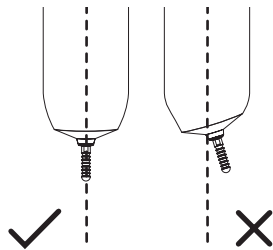

PRODUCT DESCRIPTION

The Sterling gel liner is a thermoplastic elastomer (TPE) gel liner, with fabric cover, that is available in cushion and locking options, one profile, six sizes, and three thicknesses.

INTENDED USE

The Sterling gel liner is a flexible cover that is rolled onto a limb, which creates a barrier between the skin and prosthesis. Liners provide cushion and protection and are often used as part of the suspension method for holding prosthesis onto a limb.

For Use and Target Patient Population

- Lower limb amputation and/or congenital deficiency
- No known contraindications

The device is for low to moderate impact use, e.g., walking.

TECHNICAL SPECIFICATIONS

Material	Thermoplastic Elastomer (TPE) Gel
Types	Cushion or Locking
Profiles	Uniform
Sizes	S, M, M+, L, L+, XL
Thicknesses	3 mm, 6 mm, 9 mm
Overall Length	16.5 in (42.0 cm)

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

WARNING: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

WARNING: Misuse of the device may result in loss of suspension.

CAUTION: Potential side-effects are skin irritation/reaction problems.

CAUTION: The device must not come in contact with glass, carbon fibers, or other foreign particles. Washing alone may not be sufficient to eliminate the problem. If the device is exposed to foreign substances or chemicals, it should be returned to the healthcare professional.

DEVICE SELECTION

To determine the correct size:

- Measure the circumference of the residual limb 4 and 30 cm from the distal end (*Figure 1*)
- Choose the corresponding device size. If the measurement taken does not have a corresponding device size, round the measurement down (*Figure 2*).

Note: Do not choose a device size which exceeds the measurement, i.e., do not round up.

CAUTION: A too small/tight device may result in pistoning, numbness, or blistering.

CAUTION: A too large/loose device may result in increased perspiration and movement of the limb inside the device, which may lead to blisters and rashes.

If any of the above symptoms, or any other indications of improper fit are experienced, the patient should contact the healthcare professional immediately.

FITTING INSTRUCTIONS

CAUTION: The device must be thoroughly inspected for defects before fitting.

TRIMMING INSTRUCTIONS

The device may be trimmed to increase range of motion, or for patient preference (*Figure 3*).

CAUTION: Do not cut the device below the level of the matrix or below the proximal trim-lines of the socket. Excessive trimming may reduce the suction between the device and limb, and compromise suspension.

SOCKET FITTING

Achieving the intended function of the device is dependent on the fit of the socket. Appropriate socket design is essential to achieve suspension, control, and comfort.

The following checklist should be used when checking the socket fit:

- Patient can don the socket easily without the use of lubricant aids.
- Full contact between the liner and the socket inner wall.
- No air pockets develop when the patient loads the prosthesis.
- Minimized pistoning in the socket thus, no visible movement when pulling on the prosthesis.
- Full movement possible without compromising the suspension.

CAUTION: If feedback indicates any of the above conditions are not met, the socket will need to be adjusted or remade accordingly.

CAUTION: For a cushion liner, the socket must be airtight in order to maintain suction suspension when a sleeve is applied.

CUSTOM SHAPE CAPTURE

Due to the material properties of the device, the unique shape of the limb will be automatically captured during normal use. To accelerate shape capture, “Cold Capture” or “Hot Capture” techniques can be utilized.

Both techniques require a cast/scan of the residual limb with no liner and a subsequent positive model.

COLD CAPTURE

1. Add a protective layer to the clean positive model.
2. Apply device to positive model.
3. Leave for a minimum time of 60 minutes.

HOT CAPTURE

CAUTION: Direct physical contact with heating elements will permanently damage the device.

CAUTION: Use heat-protective gloves at all times during this process.

1. Add a protective layer to the clean, dry positive model.
2. Place the positive model into a preheated oven at 65°C (150°F) for 60 minutes.
3. Apply the device to the positive model and place it into the oven.
4. Remove from the oven after a maximum of 10 minutes.
5. Allow to cool.

Note: Make sure the material of the positive model and protective layer can withstand 65°C (150°F).

Note: The device must not be put into the oven unless it is on a positive model.

LOCKING LINER

It is recommended that the device is used in conjunction with a compatible College Park lock.

CAUTION: Do not use a flangeless pin with the device (Figure 4). A flangeless pin can push through the distal attachment.

FIXED DISTAL ATTACHMENT

If a lanyard suspension system is used: Turn the connecting screw into the distal attachment of the liner. Refer to the instructions for use for the lanyard suspension system for correct instructions.

If a locking pin is used: Apply Loctite 242 or an equivalent compound to the threads of the pin and tighten the attachment pin to the fixed distal attachment, torque to 4 Nm. Refer to the instructions for use for the lock used for correct instructions.

USAGE

DONNING

CAUTION: Open wounds or damaged skin should be covered with a bandage or other appropriate covering to prevent direct contact between the wound and device.

CAUTION: Ensure the inside of the device is clean, dry, and free from any foreign objects that may cause skin irritation.

Note: Do not apply lotion to the residual limb immediately before donning the device. The residual limb should be clean and dry to prevent device damage.

LINER DONNING

1. Grip the top of the device from the inside and slide it over the hand until the inner surface is fully exposed (Figure 5). Take care not to damage the device with fingernails.
2. After exposing as much of the distal end of the device as possible, position it against the distal end of the residual limb (Figure 6).

Note: Be careful that the pin on the locking liner is positioned centrally on the distal end of the residual limb (Figure 8).

3. Bend the knee slightly.

4. With light compression, roll the device all the way upwards onto the limb (Figure 7).

CAUTION: Do not tug or pull the upper end of the liner when rolling the device upwards onto the limb. This can result in tension on the skin, causing blisters or rashes.

5. Check that no air pockets are present. If air pockets are found, reapply the device.

Note: The device must be orientated with the external labeling positioned at the front of the residual limb (Figure 9).

SOCKET DONNING

Cushion Liner: Use together with a sleeve. Refer to the instructions for use for the sleeve for correct use.

Step into the socket using body weight to achieve full engagement before using the sleeve to maintain vacuum suspension.

Locking Liner: Step into the socket using bodyweight to fully engage the distal attachment into the lock until a secure connection is achieved.

CAUTION: Ensure that the connection between the distal attachment and lock is secure and without play.

CLEANING AND CARE

Daily cleaning of the residual limb is essential. Use of a mild, pH balance, 100% fragrance- and dye-free liquid soap is recommended. If the skin is dry, apply a pH balance, 100% fragrance- and dye-free lotion to nourish and soften the skin.

Clean the device daily after use and before first use.

- Turn the device inside out (Figure 5).
- Wash with pH balanced, 100% fragrance- and dye-free liquid soap.
- After washing, rinse the device thoroughly with warm water (Figure 10), and pat dry on both sides with a lint-free cloth.
- The device shall be dried overnight before reuse.
- Always return the device to its neutral state immediately after cleaning. Do not store inside-out as device damage may result.

CAUTION: Skin health should be monitored daily. In case of abnormal symptoms, discontinue use of the device and contact the healthcare professional immediately.

CAUTION: Be aware that common household or bath products may cause or contribute to skin irritation, e.g., soaps, deodorants, perfumes, abrasive cleaners, aerosol- or alcohol sprays.

CAUTION: If device use results in excessive perspiration such that stability or suspension are compromised, it is recommended to remove the device and dry off both the device and the residual limb.

For further questions, please contact College Park Technical Service.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty inspection schedule below.

We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.

WARRANTY INSPECTION SCHEDULE FOR STERLING GEL LINER: 6 MONTHS.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

College Park does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Damage caused by component combinations that were not authorized by the manufacturer.
- Device used outside of recommended use condition, application, or environment.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc., and the competent authority of the Member State in which the prosthetist and/or patient is established.

*‘Serious incident’ is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient’s, user’s, or other person’s state of health, (c) a serious public health threat.

LIMITED WARRANTY / STERLING LINERS

College Park Industries, Inc., a Michigan corporation at 27955 College Park Drive, Warren, MI 48088 (“Company”) hereby provides its Limited Warranty for the Sterling Liner™ (a “Liner”). The Company, as set forth below, hereby provides its limited warranty, to the Liner’s original user, that the Liner is free from defects in materials and workmanship. The length of the limited warranty period depends on the type of Liner as summarized in the table below.

PRODUCT	WARRANTY PERIOD	RETURN PERIOD†	TRIAL PERIOD
Sterling Liner	6 months	90 days	30 days

†Return period is only for unopened and unused Liners.

LIMITED WARRANTY. This limited warranty is effective beginning on the date the product is sold. This limited warranty is not transferable and does not apply to experimental or research project applications. Representations, oral or written, which are inconsistent with this warranty are not authorized and, if given, should not and cannot be relied upon.

CONDITIONS. The exercise of this limited warranty is further conditioned, in part, upon the patient adequately and timely (1) reporting the claim of a defect and (2) returning the Liner to a prosthetist recognized by the Company (“Recognized Prosthetist”). The Recognized Prosthetist shall undertake, with the patient, to process that claim with the Company. Upon discovery of a possible defect, the patient shall immediately notify the Recognized Prosthetist, and, within 30 days from discovery, shall notify the Company in writing, therein describing all current claims that the patient may have with respect to the Liner.

Failure to properly give such notices shall automatically constitute full waiver of all claims and warranties by the patient and any person taking action on the patient’s behalf. If a Liner needs repair or replacement due to a defect in workmanship or materials, a Return Authorization Number (RA#) is required, which is obtained by calling the Company’s Customer Service Department at 800-728-7950. The original order date and patient name must be included with the Liner.

REPAIRS/REPLACEMENT. This Limited Warranty covers manufacturing defects on the Liner related to the gel bonding and the gel adhesion to fabric. A product returned during the warranty period will be evaluated by College Park and replaced at the discretion of College Park. The replaced Liner will continue to be covered under this Limited Warranty for the remaining duration of the warranty period that commenced on the fitting date.

There is no warranty expressed or implied, and no other liability is accepted by College Park, for damages caused by abuse, mishandling, accidents, or normal wear and tear. For example, holes from normal wear are not considered a manufacturing defect. There also is no warranty coverage for:

- Unauthorized modifications or additions to the Liner
- Liners used in sockets that were not created for a Liner
- Misuse of the product outside of the intended design format

Replacement of Liners is subject to the review and sole discretion of College Park of the actual Liner. The warranty is void if any parts of the fitting and routine care instructions stated in the Liner Technical Instructions and User Guide included with the Liner are not followed. If a Liner is returned within the warranty period and it is determined by College Park's Quality Assurance Department, in its sole discretion, to be defective, the Liner will be replaced.

DAMAGES/LIABILITIES. No person, employee, contractor, agent, distributor, dealer or company is authorized to change, modify, or extend the terms of this limited warranty in any manner whatsoever. The Company is not liable for indirect, incidental, or consequential damages in connection with the use of the Liner. These damages include, without limitation, loss, damage, or expense attributable to loss of use of the Liner, loss or damage to property other than the Liner, loss of expected income, loss from business disruption, or other commercial or personal loss due to any cause, even if the Company has been advised of the possibility of such damages, in connection with the Liner, or any use of the Liner (except for liability for consequential damages to the extent that by law such may not be disclaimed).

FORCE MAJEURE. This limited warranty will not apply if a failure(s) of the Liner is attributable to (a) neglect, (b) accident (c) misuse, (d) use for applications beyond design specifications, (e) use for purposes other than those expressed or intended by current product literature, (f) abuse, (g) modification or alteration to any component of the Liner other than in accordance with the Company's written instructions, (h) exposure of the Liner to fire, perspiration, water, corrosive materials such as salt water and PH extremes, or other contaminants or to other conditions beyond the environmental and operating constraints specified by the Company, (i) improper installation of the Liner, (j) failure to satisfy the pre-conditions or other requirements contained herein, (k) failure to strictly comply with the Company's instructions including its Technical Instructions, (l) anatomical, physiological and/or volumetric changes in the patient such as changes in the patient's weight or in the size or shape of the patient's limb, (m) use by anyone other than the original user, (n) failure to follow recommended inspection and maintenance schedules, (o) damage caused by the installation or use of parts and accessories not compatible with the Liner, or (p) any other applicable reason.

DISCLAIMER. WITH THE EXCEPTION OF THE FOREGOING EXPRESS LIMITED WARRANTY, TO THE EXTENT PERMITTED BY LAW, THE COMPANY DISCLAIMS ALL WARRANTIES, EXPRESS AND IMPLIED, INCLUDING, WITHOUT LIMITATION, STATE LAW IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THERE ARE NO EXPRESS OR IMPLIED WARRANTIES, GUARANTEES, AGREEMENTS OR SIMILAR OBLIGATIONS OF THE COMPANY, WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE OF THIS LIMITED WARRANTY. THE INVALIDITY OR UNENFORCEABILITY OF ANY PROVISION OF THIS LIMITED WARRANTY SHALL NOT AFFECT THE VALIDITY OR ENFORCEABILITY OF THE REMAINING PROVISIONS HEREOF.

RETURNS, ADJUSTMENTS, CREDITS. We stand behind our quality products and are committed to your complete satisfaction. We deliver all our prosthetic Liners with a "30 Day Satisfaction Guarantee". If you are dissatisfied with a College Park Liner for any reason, it may be returned within 30 days of the original invoice date for replacement, refund, or credit. Unopened Liners may be returned within 90 days of original invoice for replacement, refund, or credit.

All returns require a Return Authorization Number (RA#), which is obtained by calling College Park's Customer Service Department at 800-728-7950. This guarantee does not cover customer ordering errors. A restocking fee of 20% will be applied to orders returned without an RA.

©2024 College Park Industries, Inc. All rights reserved. Sterling Liners are trademarks of College Park Industries, Inc., registered in the U.S.A. and other countries.

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