
 college park

STERLING

SUSPENSION SLEEVE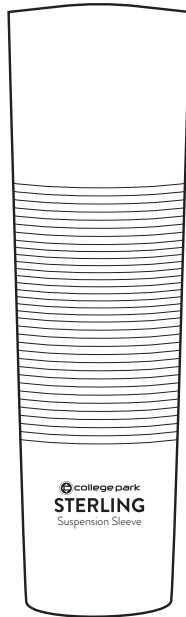

college park
TECHNOLOGY *for the* HUMAN RACE

*technical instructions,
user guide, and warranty*

FIGURE 1

FIGURE 2

FIGURE 3

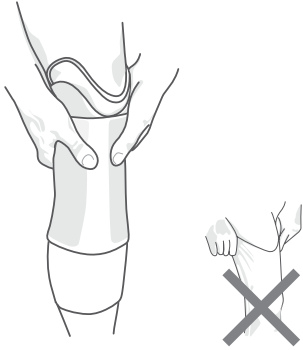
FIGURE 4

These diagrams are to help familiarize you with the unique parts of the Sterling suspension sleeve. These parts are referenced in the instructions and used when speaking with a technical service representative.

PRODUCT DESCRIPTION

The Sterling suspension sleeve is a thermoplastic elastomer (TPE) gel sleeve, with fabric cover.

INTENDED USE

The Sterling suspension sleeve is designed to aid in holding the prosthesis to the body.

For Use and Target Patient Population

- Lower limb amputation and/or congenital deficiency
- No known contraindications

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

WARNING: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

WARNING: Misuse of the device may result in loss of suspension.

CAUTION: Potential side-effects are skin irritation/reaction problems.

CAUTION: The device must not come in contact with glass, carbon fibers, or other foreign particles. Washing alone may not be sufficient to eliminate the problem. If the device is exposed to foreign substances or chemicals, it should be returned to the healthcare professional.

DEVICE SELECTION

To determine the correct size:

Measure the circumference of the residual limb 15 cm above mid-patella (Fig. 1) and choose the corresponding device size. See the sizing chart below.

SIZE	MEASUREMENT (CM)
35	35-41 cm
41	41-49 cm
49	49-55 cm
55	55-62 cm
62	62-76 cm

CAUTION

CAUTION: When cutting the device, ensure a clean cut is made. Any sharp edges or notches can lead to tears in the material and/or skin irritation.

SOCKET FITTING

CAUTION

CAUTION: The socket must be airtight in order to maintain suction suspension when the sleeve is applied.

CAUTION

CAUTION: The proximal brim of the socket must be smooth and without sharp edges that could damage the device.

USAGE

DONNING

1. Turn the device inside out, halfway, with the inner surface of the distal part half exposed.
2. Stretch the device over the brim of the socket and roll the distal end completely down (Fig. 2).
3. Roll the upper part down until the socket brim is completely exposed.
4. Align the prosthesis and step into the socket.
5. Roll the device up with both hands. Do not pull on the edges as this may place strain on the skin (Fig. 3). The device should be put on with the knee bent slightly to avoid high pressure on the patellar tendon. At its proximal end, the device shall contact the intact skin directly to obtain correct suspension performance.

**CAUTION**

CAUTION: Open wounds or damaged skin should be covered with a bandage or other appropriate covering to prevent direct contact between the wound and device.

Note: Do not apply lotion to the residual limb immediately before donning the device. The residual limb should be clean and dry to prevent device damage.

CLEANING AND CARE

Daily cleaning of the residual limb is essential. Use of a mild, pH balance, 100% fragrance- and dye-free liquid soap is recommended. If the skin is dry, apply a pH balance, 100% fragrance- and dye-free lotion to nourish and soften the skin.

Clean the device daily after use and before first use.

- Turn the device inside out.
- Wash with pH balanced, 100% fragrance- and dye-free liquid soap.
- After washing, rinse the device thoroughly with warm water and pat dry on both sides with a lint-free cloth (Fig. 4).
- Do not machine wash.
- The device shall be dried overnight before reuse.
- Always return the device to its neutral state immediately after cleaning. Do not store inside-out as device damage may result.
- The device should not be exposed to excessive heat or sunlight.
- Take care not to scrub the surface of the device. That can roughen the surface, which can cause skin irritation.

CAUTION: Skin health should be monitored daily. In case of abnormal symptoms, discontinue use of the device and contact the healthcare professional immediately.

CAUTION: Be aware that common household or bath products may cause or contribute to skin irritation, e.g., soaps, deodorants, perfumes, abrasive cleaners, aerosol- or alcohol sprays.

CAUTION: If device use results in excessive perspiration such that stability or suspension are compromised, it is recommended to remove the device and dry off both the device and the residual limb.

For further questions, please contact College Park Technical Service.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty inspection schedule below.

We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.

WARRANTY INSPECTION SCHEDULE FOR STERLING SUSPENSION SLEEVE: 3 MONTHS.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

College Park does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Damage caused by component combinations that were not authorized by the manufacturer.
- Device used outside of recommended use condition, application, or environment.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc., and the competent authority of the Member State in which the prosthetist and/or patient is established.

*‘Serious incident’ is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient’s, user’s, or other person’s state of health, (c) a serious public health threat.

LIMITED WARRANTY / STERLING SUSPENSION SLEEVES

College Park Industries, Inc., a Michigan corporation at 27955 College Park Drive, Warren MI 48088 (“College Park”), hereby provides a Limited Warranty for each Sterling Suspension Sleeve from the delivery date of the initial purchase, for a period of ninety (90) days, against defects in material and workmanship as defined in the following paragraphs.

PRODUCT	WARRANTY PERIOD	RETURN PERIOD
Sterling Suspension Sleeve	90 days	90 days

LIMITED WARRANTY. This Sterling Suspension Sleeve Limited Warranty is effective beginning on the date the product is initially delivered to the end user. This Limited Warranty is not transferable.

CONDITIONS. This Limited Warranty covers manufacturing defects on the Sterling Suspension Sleeve related to the gel bonding and the gel adhesion to fabric. A product returned during the warranty period will be evaluated by College Park and replaced at the discretion of College Park. The replaced Sterling Suspension Sleeve will continue to be covered under this Limited Warranty for the remaining duration of the warranty period that commenced on the fitting date. There is no warranty expressed or implied, and no other liability is accepted by

College Park, for damages caused by abuse, mishandling, accidents, or normal wear and tear. For example, holes from normal wear are not considered a manufacturing defect. There also is no warranty coverage for:

- Unauthorized modifications or additions to the Sterling Suspension Sleeve
- Misuse of the product outside of the intended design format

Replacement of Sterling Suspension Sleeves is subject to the review and sole discretion of College Park of the actual sleeve. The warranty is void if any parts of the fitting and routine care instructions stated in the Sterling Suspension Sleeve Technical Instructions included with the sleeve are not followed. If a Sterling Suspension Sleeve is returned within the warranty period and it is determined by College Park's Quality Assurance Department, in its sole discretion, to be defective, the Sterling Suspension Sleeve will be replaced.

RETURNING A PRODUCT: All returns require a Return Authorization Number (RA#), which is obtained by calling College Park's Customer Service Department at 800-728-7950. This guarantee does not cover customer ordering errors. A restocking fee of 20% will be applied to orders returned without an RA#.

DAMAGES/LIABILITIES. No person, employee, contractor, agent, distributor, dealer or company is authorized to change, modify, or extend the terms of this Limited Warranty in any manner whatsoever. College Park is not liable for indirect, incidental, or consequential damages in connection with the use of the Sterling Suspension Sleeve. These damages include, without limitation, loss, damage, or expense attributable to the loss of use of the Sterling Suspension Sleeve, loss or damage to property other than the Sterling Suspension Sleeve, loss of expected income, loss from business disruption, or other commercial or personal loss due to any cause, even if College Park has been advised of the possibility of such damages, in connection with the Sterling Suspension Sleeve, or any use of the Sterling Suspension Sleeve (except for liability for consequential damages to the extent that by law such may not be disclaimed).

DISCLAIMER: WITH THE EXCEPTION OF THE FOREGOING EXPRESS LIMITED WARRANTY, TO THE EXTENT PERMITTED BY LAW, COLLEGE PARK DISCLAIMS ALL WARRANTIES, EXPRESS AND IMPLIED, INCLUDING, WITHOUT LIMITATION, STATE LAW IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THERE ARE NO EXPRESS OR IMPLIED WARRANTIES, GUARANTEES, AGREEMENTS OR SIMILAR OBLIGATIONS OF COLLEGE PARK, WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE OF THIS LIMITED WARRANTY. THE INVALIDITY OR UNENFORCEABILITY OF ANY PROVISION OF THIS LIMITED WARRANTY SHALL NOT AFFECT THE VALIDITY OR ENFORCEABILITY OF THE REMAINING PROVISIONS HEREOF. REPRESENTATIONS, ORAL OR WRITTEN, WHICH ARE INCONSISTENT WITH THIS WARRANTY ARE NOT AUTHORIZED AND, IF GIVEN, SHOULD NOT AND CANNOT BE RELIED UPON.

©2024 College Park Industries, Inc. All rights reserved. Sterling Suspension Sleeve is a registered trademark of College Park Industries in the U.S.A.

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